

TRIACARP JUNIOR
Fixed wrist hand brace
Ref: SRT 222



INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests.

These orthoses meet the essential requirements of Government's Decision 54/2009 and have registration Certificates from the Ministry of Health.

These products satisfy the European Medical Device Directive 93/42/CEE (RD.1591/2009).

The orthosis is used according to the recommendations of the specialized physician. The product must be adjusted by the orthopaedic technician!

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

Available sizes, left/right versions

Sizes	1	2	3
Wrist circumference (cm)	10-12	12-14	14-16
Length (cm)	18	20	22
Reference	SRT222M1	SRT222M2	SRT222M3

After the code must be added **L for Left, R for Right**

The circumference of the wrist is measured

Indications

It is used in sprains, dislocations, carpal tunnel syndrome, radial nerve palsy and fractures at the level of the area, both as orthopedic and postoperative treatment.

Description

It is made of textile, provided with:

- an anatomically shaped, mouldable metal palm splint
- extra strip around the wrist for a better fit.

Hook and loop fastener on the dorsal side of the hand. Length varies between 14-18 cm, depending on the size.

The orthosis stabilizes and immobilizes the radiocarpal joint.

Composition

The orthosis is made of a three-layer laminated textile:

- **Asterix** - The polyamide fiber has a high resistance to chemical and biological agents, resists abrasion and does not absorb water.
- **Moltopren** (MTP) - air-permeable open-cell polyurethane foam (airflow in both directions is inversely proportional to foam density) with

high moisture absorbing properties.

- **Jersey cotone SOFT** - 100% cotton or cotton with spandex - elastane (for increased elasticity).

SOFT

The code for the balsamic treatment of the cotton fabric to provide softness to the touch and to avoid skin irritation.

Positioning

Place the orthosis so that the metal splint supports the palm, then close the orthosis on the dorsal forearm, secure the strip around the wrist and adjust to the optimum compression.

Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

The legal guarantee of conformity according to Law no. 449/2003: **2 years.**

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON	
30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller signature and stamp



Manufacturer: TRIAMED SRL

România/Timiș/Timișoara, Str. Calea Buziașului nr.77, Hala 1-C1/A

Tel/Fax: 00 40 256 487388, Tel: 00 40 765 240037

office@triamed.ro, triamed@gmail.ro

www.triamed.ro