

## PROCONFORT

Lumbosacral orthosis  
Ref. SRT 102

### INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

#### Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

#### Available sizes

Sizes	1	2	3	4	5	6
Waist circumference (cm)	65-75	75-85	85-95	95-105	105-115	115-125
Reference	SRT102M1	SRT102M2	SRT102M3	SRT102M4	SRT102M5	SRT102M6

The circumference of the waist is measured

#### Indications

It is used in case of incipient intervertebral disc herniation, moderate low back pain, postoperative, in case of osteoporosis, for moderate stabilization of the lumbar spine.

#### Description

The orthosis is made of elastic fabric, provided with:

- 4 plastic rods located on the posterior component, to support the lumbar region
- 2 plastic rods located on the anterior component, to support the abdominal region
- 2 elastic straps of 6 cm around the waist, providing stability and compression
- a semirigid pad made of polyurethane foam, with textile liner, which produces a micromassage effect at lumbar level and helps to preserve local heat and protect the area. The shape and pressure points of the pad achieve a massage effect in the lumbar area. The lumbar pad is removable and can be positioned as needed.

Hook and loop fastener on the front side.

Width: front 16 cm, back 27,5 cm.

For an easy application it is provided with helping strips.

The 6 plastic rods stabilize the lumbar region, slightly reducing lordosis, and the compression exerted by the elastic bands stimulates local metabolism, reducing muscle tension and pain.

The semirigid pad included in the orthosis, provides protection and compression to the lumbar region. It has hook and loop fastener and it is removable, and can be positioned as needed.

It is a traditional elastic corset that, due to the material, allows normal thoraco abdominal breathing, while ensuring uniform compression on the entire covered surface.

#### Purpose

The orthosis stabilizes the lumbar region, slightly reducing lordosis, the compression exerted by the elastic bands stimulates local metabolism, reducing muscle tension and pain.

#### Composition


Elastic fabric with a structure that allows high air permeability in both directions.

#### Positioning

The orthosis is applied from back to front, the wider part is fixed on the lumbar region of the back, the narrow part is closed in the front, on the abdomen. The double elastic strap of 6 cm surrounds the waist. The lumbar part is perfectly stretched, without creases, at a correct fixation.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**

#### Cleaning

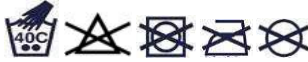

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

#### Storage

At room temperature, in a dry space with good ventilation.

**Commercial warranty of the product: 30 days.**

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

<b>WARRANTY COUPON</b> 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016



**Manufacturer SC TRIAMED SRL**

**Production** Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

**Registered office** Str. Cosminului nr.12 | Timișoara | Timiș | România

**Phone** 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025