

## TRIA SAB

Arm abduction orthosis to 45°/60°

Ref. SRT 221

### INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

#### Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

#### Available sizes

Sizes	1	2	3
Arm and thorax circumference (cm)	85-105	105-125	125-145
Arm and palm length (without fingers) (cm)	36	39	43
Reference	SRT221M1	SRT221M2	SRT221M3

The circumference of the arm and thorax is measured. Arm and palm length (without fingers) is measured

The orthosis can be used on both the left and right arm.

#### Indications

It is used in plastical surgery, orthopedics, postoperative, post traumatical arm and shoulder injuries, scapulohumeral fractures.

#### Description

The orthosis has 3 components:

- arm support cushion, made of polyurethane foam (sponge), covered with textile material, provided with two inclinations: 45° and 60°. The abdominal cushion is fixed to the abdomen with a positioning belt provided with 2 double buckles for adjustment
  - forearm fixing component, made of polyurethane foam (sponge)
  - dynamic hand support component, optional (cylinder made of low-density polyurethane foam, for fist gymnastics)
  - thoracic harness made of 4 straps adjustable by cutting them and repositioning the "crocodile" type hook and loop fastening, provided with 2 rings and 2 protective sleeves.
- The orthosis has the possibility of fixing the arm in abduction at 45° or 60° from the chest, by changing the position of the support cushion.

It can be used both on the left and on the right side.

Hook and loop fastener.

#### Purpose

The orthosis stabilizes the shoulder joint in abduction, preventing unwanted movements and promoting healing .

#### Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

#### Positioning

The arm support cushion is fixed between the chest and the arm so that the abduction angle of the arm corresponds to the instructions of the specialist. The harness is fixed on the sternum, one long side of the harness is passed over the healthy shoulder and the other subaxillary and both are fastened on the back side of the cushion. It is fixed to the optimum length by cutting the strips and repositioning the "crocodile" type hook and loop fastening. The short straps of the harness are fixed on the front of the cushion. The arm cushion and the one for the fist are fixed in such a way that they allow the forearm and palm to rest on the cushion.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**



#### Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

#### Storage

At room temperature, in a dry space with good ventilation.

**Commercial warranty of the product: 30 days.**

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.


**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

<b>WARRANTY COUPON</b> 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016

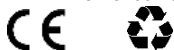

**Manufacturer SC TRIAMED SRL**

**Production** Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

**Registered office** Str. Cosminului nr.12 | Timișoara | Timiș | România

**Phone** 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/28.07.2025