

ELBI PREMIUM PLUS

Epicondylitis orthosis

Ref. SRT 216

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Sizes	1	2	3	4	5	6
Elbow circumference (cm)	20-23	23-26	26-29	29-32	32-35	35-38
Reference	SRT216M1	SRT216M2	SRT216M3	SRT216M4	SRT216M5	SRT216M6

The circumference of the elbow is measured

The orthosis can be used on both the left and the right hand

In order to be used on the left hand, the pad must be repositioned.

Indications

It is used in epicondylitis, arthrosis, post fracture rehabilitation of the elbow, elbow joint overstraining (tennis elbow).

Description

Orthosis made of textile, provided with:

- detachable pressing pad at the epicondyle, that exerts pressure on the ligaments
- detachable fastening strap with buckle, for adjusting the compression on the pad.

Hook and loop fastener.

Length: 28 cm.

The orthosis is a support for the elbow and can be used both as orthopedic treatment in minor diseases and for recovery in the post immobilization period. Due to the fastening strap with buckle, it can be adjusted to the optimal compression and can be used in sport activities that overload the elbow, in case of epicondylitis.

The orthosis is factory assembled for the right hand. To use it on the left hand, both the pad and the strap are repositioned. For effective compression, the strap with the buckle is positioned so that the buckle overlaps the pad. By removing the strap and the pad, the orthosis becomes a mobile elbow orthosis.

Purpose

The orthosis stabilizes and immobilizes the elbow.

Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The orthosis is placed on the elbow and the straps are closed on the internal side of the arm. The band with buckle is positioned on the upper third of the forearm, at the level of the pad, to the optimal compression.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: supportclient@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016



Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/28.07.2025