

## TRIAGEN PLUS

Mobile knee orthosis with lateral articulations

Ref. SRT 343

### INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

#### Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

#### Available sizes

Sizes	1	2	3	4	5	6	7	8
Knee circumference (cm)	33-35	35-37	37-39	39-41	41-43	43-45	45-47	47-49
Reference	SRT343M1	SRT343M2	SRT343M3	SRT343M4	SRT343M5	SRT343M6	SRT343M7	SRT343M8

The circumference of the knee is measured at the middle of the knee cap

The orthosis can be used on both the left and the right knee.

#### Indications

It is used in lateral instability of the knee, traumas or injuries at the level of the knee, inflammatory processes, for postoperative recovery.

#### Description

Orthosis made of textile, open in the front, provided with:

- a non-slip silicone elastic band (10mm) inside the upper part
- 2 straps for fixation provided with elastic hook and loop fastener, that mold to the conical anatomy of the thigh and calf, preventing the orthosis from slipping
- 2 metal splints with mobile, adjustable and lockable articulations, one on each side. The adjustment device is easily accessed on the sides, covered with a plastic cap. The angle is adjusted with 2 rivets. The possibilities of limiting the movement are:

- flexion 0°, 10°, 20°, 30°, 60°, 90°

- extension 0°, 10°, 20°, 30°, 60°, 90°

- cutout in the popliteal region for comfort.

Hook and loop fastening on the front side.

Length: 33 cm.

Due to its particular shape, it can be used also in case of a deformed knee. The compression can be adjusted by the frontal closure system and the 2 fastening straps.

#### Purpose

The orthosis is a support for the knee joint, the 2 metal splints with mobile articulation ensuring stability and safety in use.

#### Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

#### Positioning

The orthosis is placed on the knee making sure that the knee cap is positioned correctly, in the middle of the orthosis and that it remains free. The 2 elastic straps are fixed to the optimal compression.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**

#### Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

#### Storage

At room temperature, in a dry space with good ventilation.

#### Commercial warranty of the product: 30 days.

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.


#### Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

WARRANTY COUPON	
30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016

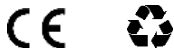

**Manufacturer SC TRIAMED SRL**

**Production** Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

**Registered office** Str. Cosminului nr.12 | Timișoara | Timiș | România

**Phone** 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025