

TRIANORTH

Fixed shoulder elbow wrist hand orthosis

Ref. SRT 234

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Universal size, for thorax and arm circumference between 90 – 150 cm.

The orthosis can be used on both the left and the right shoulder.

Indications

It is used to support the arm and forearm, in case of humerus sprain, humerus collar fracture, the humeral tuberosity fracture without movement, shoulder blade fracture, scapulohumeral dislocation.

Description

Orthosis made of textile with 3 components that fix the arm and the forearm, provided with one support strap for the orthosis and one support strap for the thumb. The main component is positioned at the thoracoabdominal level and the other 2 (which fix the arm and the forearm) are positioned with the help of the hook and loop fastening, on the main component. The main component and the one that fixes the arm are adjustable. The regulation is done by cutting at the level of the stitches to avoid fabric breaking. The adjustable shoulder strap can be adjusted by cutting it and repositioning the "crocodile" type hook and loop fastening.

It is provided with:

- one double buckle for adjusting the length of the belt

- shoulder protection sleeve.

Hook and loop fastener.

Purpose

The orthosis is a support for the arm and forearm. Due to the 3 components and to the support strap, does not allow the abduction of the arm and external rotation.

Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The main component of the orthosis is positioned at the thoracoabdominal level and is fixed with the hook and loop fastening. The forearm is positioned at an angle of 90° to the affected arm, and the other 2 components (which secure the arm and forearm) are positioned on the main component. It is adjusted by cutting to the appropriate size to obtain the optimum compression.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**

Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016



Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/28.07.2025