

TRIAFIX ARM 02

Arm support
Ref. SRT 235

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The support is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the support.

This support should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Universal size, for a forearm length between 22 – 40 cm.

The support can be used on both the left and the right arm.

Indications

It is used to support the arm and forearm.

Description

The support is made of one single textile piece that supports the arm, 2 adjustable support straps and one fastening strap for the thumb. The two support straps can be adjusted by cutting them and repositioning the "crocodile" type fastener.

The adjustable shoulder strap is provided with:

- one Lock double buckle for adjusting the length of the belt
- shoulder protection sleeve.

The waist support belt can be removed.

Hook and loop fastener.

Purpose

The support is an efficient support for the arm and forearm. Due to the two fastening straps, it does not allow the arm abduction and external rotation.

Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The arm is positioned in the support at a 90° angle and the bands are fastened at the desired height. The waist support belt can be removed.

If it is necessary to adjust the position of the support, the adjustment is made by fully opening the hook and loop fastener and repositioning the support higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the support while is fixed.**



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.


Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: supportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016

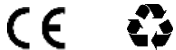

 **Manufacturer SC TRIAMED SRL**

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/28.07.2025