

CERVITEX JUNIOR

Soft cervical orthosis

Ref. SRT 125

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes in 2 different heights: 6 and 8 cm

Sizes	1	2	3
Neck circumference (cm)	21-24	24-27	27-30
Height	6 and 8	6 and 8	6 and 8
Reference	SRT125M1H	SRT125M2H	SRT125M3H

After the code must be added **the height in cm: 6 or 8**

The circumference of the neck is measured

Indications

It is used in case of cervical injuries, torticollis, light cricks, post surgery on the cervical spine.

Description

The orthosis is made of a 3 cm wide polyurethane foam (sponge) with cotton covering. It is anatomically shaped for optimal fit and increased comfort.

Hook and loop fastener on the back side.

Due to the anatomical shape and to the density of the foam, the soft cervical orthosis limits the flexion and extension movements of the cervical spine. This provides support for the cervical spine, helping the healing process.

Purpose

The soft cervical orthosis provides support for the cervical spine, helping the healing process.

Composition

Sponge made of Polyurethane foam.

The fabric (covering) is soft and it does not cause irritation. It allows a good ventilation, it absorbs and easily removes moisture from the body, keeping the body dry.

Positioning

The orthosis is placed under the chin, so that the chin will comfortably rest in the anatomical indentation and is fixed at the back with hook and loop closer straps.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.


In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015
SR EN ISO 13485:2016

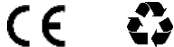

 **Manufacturer SC TRIAMED SRL**

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025