

HERNIA DUO

Hernia orthosis
Ref. SRT 151

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Universal size

For hip circumference between 75 – 135 cm.

Indications

It is used in case of small or medium reducible inguinal hernia.

Description

Orthosis with anatomical design, made of textile material, provided with:

- 2 leg straps

- 2 pressure pads, individually removable.

Hook and loop closure on the side. The hip band is provided on one end with "crocodile" type fastener, which allows the adjustment of the length by cutting it.

The 2 groin fixation straps are also adjustable by cutting, provided with "crocodile" type fastener.

The orthosis is small and can be easily worn under the clothes.

The bilateral inguinal hernia orthosis has the advantage of individual positioning of each pad thanks to the hook and loop fastening and at the same time, it offers the possibility of removing one of the pads, thus becoming unilateral. The strap on the unused side can be cut off and the orthosis can be used on the left or on the right.

The orthosis provides support and comfort in case of reducible inguinal hernia.

Purpose

The orthosis provides support and compression in the groin area, either to prevent the occurrence or recurrence of a hernia, or as an aid in the postoperative period.

Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Elastic fabric with a structure that allows high air permeability in both directions.

Polyethylene foam (compression pad).

Positioning

The orthosis is positioned from front to back, it is closed laterally with the straps provided with "crocodile" fasteners. The length of the straps can be adjusted in advance.

The front fixing straps are passed behind and adjusted so that the pads are well fixed. If both pads are not required, the useless one should be removed from the beginning.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.


Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON	
30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016

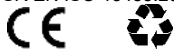

Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025