

## THORATEX 01

Elastic thoracic orthosis

Ref. SRT 112

### INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

#### Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

#### Available sizes

Sizes	1	2	3	4	5	6
Torso circumference (cm)	75-85	85-95	95-105	105-115	115-125	125-135
Reference	SRT112M1	SRT112M2	SRT112M3	SRT112M4	SRT112M5	SRT112M6

The circumference of the torso is measured

#### Indications

It is used to correct vicious positions of the thoracic spine, poor postural muscle tone.

#### Description

The orthosis is made of elastic fabric.

It has 2 components that go around the shoulders and are fixed on the chest, correcting the position of the torso by traction.

Hook and loop fastening on the front side.

Width: 6 cm.

Due to the 2 components and the frontal closure, the positioning can be done easily.

#### Purpose

The orthosis provides support and stability to the thoracic spine, corrects vicious positions of the thoracic spine, kyphosis. By wearing the brace, the wearer becomes more aware of his posture and is helped to maintain it correctly.

#### Composition

The elastic fabric is soft and it does not cause irritation. It allows a good ventilation, it absorbs and easily removes moisture from the body, keeping the body dry.

#### Positioning

The 2 bands are passed around the shoulders, crossed on the back and fastened in the front, on the thorax.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**

#### Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

#### Storage

At room temperature, in a dry space with good ventilation.

#### Commercial warranty of the product: 30 days.

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.


In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [supportclienti@triamed.ro](mailto:supportclienti@triamed.ro)

WARRANTY COUPON	
30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015  
SR EN ISO 13485:2016

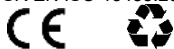

**Manufacturer SC TRIAMED SRL**

**Production** Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

**Registered office** Str. Cosminului nr.12 | Timișoara | Timiș | România

**Phone** 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025