

TRIAMED

MATERNITY

Back support
Ref: SRT 103



INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here.

The product is intended for single patient use.

No liability is accepted for incorrect product use.

Guarantee is bound to statutory regulations.

Available sizes

Universal size

For hip circumference between 80 – 140 cm.

Indications

It is used during pregnancy giving moderate stability to the abdomen and the lumbar spine. To be used only with medical recommendation.

Description

The support has 2 components:

- the lumbar part made of elastic fabric, provided with 4 integrated plastic rods
- the frontal part made of soft, rigid fabric.

It supports the abdomen and it is adjustable so it can be used for the whole period of the pregnancy.

Hook and loop fastener on the front side.

Width: front 11 cm, back 16 cm.

For an easy application it is provided with helping strips.

The anterior rigid component provides support to the abdomen, the elastic back component stabilizes the lumbar region.

Composition

Elastic fabric: 50% polyester, 33% polyamide, 17% elastane, with a structure that allows high air permeability in both directions.

Positioning

The orthosis is placed from the back to the front, positioning the front component so that it provides ample space to the abdomen.



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

Manufacturer

SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Uliuc 262C | Sacoșu Turcesc | Timiș | România

Phone 00 40 256 48 73 88

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 3, Revision 3/23.10.2023