

TRIADES JUNIOR

Fixed shoulder elbow wrist hand orthosis

Ref. SRT 225

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes, left/right versions

Sizes	1	2	3
Arm and thorax circumference (cm)	60-70	70-80	80-90
Reference	SRT225M1	SRT225M2	SRT225M3

The circumference of the arm and thorax is measured

The orthosis can be used on both the left and right shoulder.

Indications

It is used to immobilize the arm and forearm in case of sprains, luxations or fractures of the scapulohumeral joint, contusions, for postoperative immobilization. It is the equivalent of Dessault's bandage.

Description

The orthosis is made of textile, one single component surrounding the torso and the shoulder, together with arm and forearm. The textile material has two faces, the orthosis being able to be used on both limbs by turning backwards and repositioning the hook and loop fasteners.

The orthosis is a support for the forearm, arm and shoulder. Due to its particular shape and the anatomical surface which it covers, it does not allow arm abduction and external rotation.

Purpose

The orthosis is a support for the forearm, arm and shoulder, it does not allow arm abduction and external rotation.

Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The orthosis is fastened on the affected shoulder, is passed under the opposite armpit and is secured in the back. The affected arm is flexed at a 90° angle, the arm component is lifted and fastened on the thoracic part. The 2 rectangular bands, fitted with hook, fasten the arm and the elbow. If required, readjust the orthosis on the back. If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.


Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON	
30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016

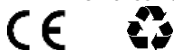

Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/28.07.2025