

TRIAFLIP

Arm rotation orthosis to 90°

Ref. SRT 232

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Universal size for circumference of arm and thorax between 90 – 150cm

The orthosis can be used on both the left and the right arm.

Indications

It is used in plastical surgery, orthopedics, postoperative, post traumatical arm and shoulder injuries, scapulohumeral fractures.

Description

The orthosis has 2 components:

- arm rotation cushion made of polyurethane foam, covered with fabric. The cushion is fixed to the abdomen by a positioning belt provided with one double buckle for adjustment
- dynamic hand support component, optional (cylinder made of lowdensity polyurethane foam, for fist gymnastics)
- arm support made of one single piece that holds the arm on the cushion, provided with a support strap for the thumb and a thoracic harness. This is made of 4 straps adjustable by cutting them and repositioning the "crocodile" type hook and loop fastening, provided with 2 rings and 2 protective sleeves.

The orthosis positions and secures the arm in external rotation at 90° to the front plan of the body.

It can be used both on the left and the right side.

Hook and loop fastener.

Purpose

The orthosis stabilizes the shoulder joint in abduction, preventing unwanted movements and promoting healing .


Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

Fix the cushion on the anterolateral side of the abdomen with a positioning belt, provided with a buckle for adjustment. Position the arm in the arm support at the optimal height, with the help of the support harness, by cutting the bands and repositioning the "crocodile" type fastening. The support strap for the thumb is optional.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**

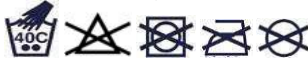

Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.


Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016

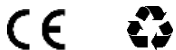

Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/28.07.2025