

THORATEX 02 JUNIOR

Rigid thoracic orthosis

Ref. SRT 126

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Universal size for thorax circumference between 45 - 75 cm.

Indications

It is used to correct vicious positions of the thoracic spine, or in case of broken collarbone.

Description

The orthosis is made of soft textile, which does not stretch (rigid) and does not harm while wearing, provided with:

- one posterior component that supports the thoracic spine
- 2 rigid straps around the shoulders, correcting the body posture by traction. The 2 straps are positioned over the head of the humerus, inserted into the 2 buckles and fastened in the front, on the thorax. They are adjustable by cutting. One of the strap is provided with "crocodile" type fastening. To shorten the straps, remove the "crocodile" type fastening from the extremity, cut the strap to the optimum length on the prefigured transverse marks and reposition the hook and loop on one of the straps. The big surface of the hook and loop fastening allows a good adjustment and strong traction, helping the clavicle to return to its anatomical shape after fracture.

The closure on the chest ensures effective fixation of the brace and prevents rotation and movement around the shoulder. For a better result it is recommended to readjust during wearing.

Purpose

The orthosis provides support and stability to the thoracic spine, corrects the vicious positions of the thoracic spine.

It stabilizes and aligns the clavicle fracture and helps the healing process.

Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The orthosis is positioned on the dorsal segment of the spine, the straps are passed around the shoulders, inserted into the 2 buckles and then it is fastened in the front, on the thorax.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.


In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015
SR EN ISO 13485:2016

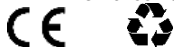

 **Manufacturer SC TRIAMED SRL**

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025