

TRIAGEN EXTRA

Mobile knee orthosis with patellar support and lateral articulations
Ref. SRT 341

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Sizes	1	2	3	4	5	6	7	8
Knee circumference (cm)	33-35	35-37	37-39	39-41	41-43	43-45	45-47	47-49
Reference	SRT341M1	SRT341M2	SRT341M3	SRT341M4	SRT341M5	SRT341M6	SRT341M7	SRT341M8

The circumference of the knee is measured at the middle of the knee cap

The orthosis can be used on both the left and the right knee.

Indications

It is used in lateral instability of the knee, traumas or injuries at the level of the knee, inflammatory processes, for postoperative recovery.

Description

Orthosis made of textile, open in the front, provided with:

- a non-slip silicone elastic band (10mm) inside the upper part
- 2 hook and loop straps on the thigh
- 2 hook and loop straps on the leg
- 2 metal splints with mobile articulations, one on each side
- cutout in the popliteal region for comfort.

At the level of the opening for the knee cap, the textiles are overlapped, and after the orthosis is closed, a circular patellar support is created.

Length: 38 cm.

Due to its particular shape, it can be used also in case of a deformed knee.

The compression can be adjusted by the frontal closure system and the 4 fastening straps

Purpose

The orthosis prevents knee cap traumas and knee articulation traumas.

Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The orthosis is placed on the knee from the back to the front, making sure that the knee cap is positioned correctly, in the middle of the orthosis and that it remains free. The 4 straps are fixed to the optimal compression.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**

Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016



Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025