

THIGH BELT

Thigh prosthesis belt

Ref. CPC 001

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The belt is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the belt.

This belt should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes, left/right versions

Sizes	1	2	3	4
Circumference of the thigh (cm)	40-51	48-59	56-67	64-75
Circumference of the hip (cm)	84-100	94-110	104-120	114-130
Reference	CPC001M1	CPC001M2	CPC001M3	CPC001M4

After the code must be added **L for Left, R for Right**

The circumference of the thigh is measured at the proximal third

The circumference of the hip is measured

Indications

It is used to fix the thigh prosthesis.

Description

It is made of textile and confers extra safety for patients using a thigh prosthesis.

The circular component of the belt, which is applied to the socket, is made of raw neoprene ("shark skin"), with anti-slip and anti-rotation properties.

Hook and loop fastening on the front side.

Purpose

The belt prevents the prosthesis from slipping and rotating on the amputation stump, ensuring stability and safety in use.

Composition

Neoprene coated on both sides with fabric.

Positioning

The belt is placed on the prosthesis. After the prosthesis is fixed on the limb, the belt is fastened on the waist.

If it is necessary to adjust the position of the belt, the adjustment is made by fully opening the hook and loop fastener and repositioning the belt higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the belt while is fixed.**

Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON	Batch and date of manufacture
30 days from the date of purchase	Date of purchase
Product reference	Seller name and signature
The reason of complaint	

SR EN ISO 9001:2015
SR EN ISO 13485:2016



 **Manufacturer SC TRIAMED SRL**

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025