

THORATEX 05

Semirigid thoracic support

Ref: SRT 142



INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests.

These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

Available sizes

Sizes	1	2	3
Torso circumference (cm)	75-95	95-115	115-135
Reference	SRT142M1	SRT142M2	SRT142M3

The circumference of the torso is measured

Indications

It is used to correct vicious positions of the thoracic spine, or in case of kyphosis.

Description

The support is made of soft textile, which does not harm while wearing, provided with:

- one posterior component that supports the thoracic spine, made of rigid fabric (which does not stretch)
- 2 straps around the shoulders, correcting the body posture by traction. The 2 straps have one elastic and one rigid component, are positioned over the head of the humerus and are adjustable by cutting. To shorten the straps, remove the "crocodile" type fastening from the extremities, cut the strap to the optimum length on the prefigured transverse marks and reposition the hook and loop.
- 2 paravertebral integrated metal rods.

Hook and loop fastener on the front side.

Composition

Two and three-layer laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The orthosis is positioned on the dorsal segment of the vertebral column and it is fastened in the front, on the thorax. The 2 straps are passed around the shoulders, crossed on the back and fastened in the front, on the thorax.



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

The legal guarantee of conformity according to Law no. 449/2003: **2 years.**

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller signature and stamp



Manufacturer: TRIAMED SRL

România/Timiș/Timișoara, Str. Calea Buziașului nr.77, Hala 1-C1/A

Tel/Fax: 00 40 256 487388

office@triamed.ro, triamed@gmail.ro

www.triamed.ro

Edition 3, Revision 2/07.11.2022