

## TRIAFIX M FINGER

Fixed wrist hand finger brace

Ref. SRT 230

### INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

#### Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

#### Available sizes, left/right versions

Sizes	1	2	3	4	5
Wrist circumference (cm)	15-17	17-19	19-21	21-23	23-25
Reference	SRT230M1	SRT230M2	SRT230M3	SRT230M4	SRT230M5

After the code must be added **L** for Left, **R** for Right

The circumference of the wrist is measured

#### Indications

It is used in sprains, dislocations, arthrosis, carpal tunnel syndrome, radial nerve paralysis, tenosynovitis, fractures at the level of the area, both as orthopedic and postoperative treatment.

#### Description

Brace made of stiff, breathable textile, provided with:

- one mouldable metal palm splint
- one mouldable metal thumb splint
- 2 inserted plastic splints on the dorsal side of the hand.

The closure is made with laces, using a strip with hook and loop, on the back of the hand.

Length: 22 cm.

For the easiest application of the orthosis, the laces have a common fastening system. The orthosis is a semirigid medical device that stabilizes and immobilizes the radiocarpal joint, limiting flexion and extension, while the thumb metal splint immobilizes the thumb.

#### Purpose

The orthosis stabilizes and immobilizes the radiocarpal joint, limiting flexion and extension, while the thumb metal splint immobilizes the thumb.

#### Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

#### Positioning

The orthosis is positioned so that the metal splint supports the palm. The orthosis is fastened on the back side of the forearm to the optimal compression.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while it is fixed.**



#### Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

#### Storage

At room temperature, in a dry space with good ventilation.

#### Commercial warranty of the product: 30 days.

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.


#### Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

<b>WARRANTY COUPON</b> 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016

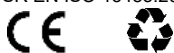

**Manufacturer SC TRIAMED SRL**

**Production** Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

**Registered office** Str. Cosminului nr.12 | Timișoara | Timiș | România

**Phone** 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/28.07.2025