

TRIAFIX GENU JUNIOR

Fixed knee orthosis

Ref. SRT 332

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Sizes	1	2	3
Length of the orthosis (cm)	35	40	45
Thigh circumference	22-42	24-46	26-50
Calf circumference (cm)	18-28	20-30	22-32
Reference	SRT332M1	SRT332M2	SRT332M3

The circumference of the thigh is measured on the upper 1/3, and the circumference of the calf on the lower 1/3

The orthosis can be used on both the left and the right knee.

Indications

It is used to immobilize and stabilize the knee in case of cricks, patellar luxations, patellar fractures without displacement, inflammatory processes.

Description

The orthosis is made of textile and consists of 3 pieces which allow width adjustment, provided with:

- a non-slip silicone elastic band (10mm) inside the upper part
 - 4 straps for fastening on the front (2 on the thigh, 2 on the leg)
 - 2 straps for fastening on the back (one on the thigh, one on the leg)
 - 4 mouldable metal splints (2 on the sides and 2 on the back).
- Hook and loop fastening.

Purpose

The orthosis immobilizes and stabilizes the knee joint.

Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The orthosis is placed from back to front, making sure that the knee cap is positioned correctly, in the middle of the orthosis and it remains free. If it is required, the side pieces are positioned again. All the straps are fixed to the optimal compression, so that the orthosis doesn't slip down.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while it is fixed.**



Cleaning

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.


Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016

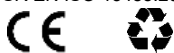

Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025