

ANKLEFIX SPORT

Mobile ankle orthosis with metal splints

Ref. SRT358

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Universal size for ankle with circumference between 20 - 28 cm.

The orthosis can be used on both the left and right ankle.

Indications

It is used in cricks, luxations, inflammatory processes, post consolidated fractures, after the removal of the orthopedic plaster cast.

Description

Orthosis with anatomical design, made of textile, provided with:

- 2 elastic straps crossed around the ankle for compression and fastening, which can be repositioned as needed (the 2 elastic straps are provided with 2 pockets for the metal splints. These elastic straps can be removed or repositioned according to ankle circumference)
- 2 metal side splints, one on each side
- 2 flexible metal splints, one on each side. For a perfect fit at the malleolus, the 2 splints from both sides can be switched (the metal one with the flexible one)

The heel area is open.

Hook and loop fastener on the front side.

Height: 23 cm.

Purpose

The orthosis is an ankle support, the 2 flexible metal splints and the 2 metal side splints ensure stability and safety in use. The crossed straps around the ankle and around the 4 splints exert additional compression, making the orthosis an efficient rigid support.

Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The orthosis can be used on both limbs and is packed for right side use. To use the orthosis on the left side, please switch on both sides the insertion order of the metal splint and the flexible splint. The orthosis is placed on the foot so that the straps are fastened in the front and the long bands are crossed under the sole and are fastened in the back to the optimal compression.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**

Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.


Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

| | |
|---|-------------------------------|
| WARRANTY COUPON 30 days from the date of purchase | Batch and date of manufacture |
| Product reference | Date of purchase |
| The reason of complaint | Seller name and signature |

SR EN ISO 9001:2015

SR EN ISO 13485:2016

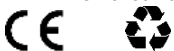

Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului.nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025