

TRIAFIX CARDIO ST

Thoracic orthosis

Ref. SRT 105

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

Sizes	1	2	3	4	5	6
Torso circumference (cm)	75-85	85-95	95-105	105-115	115-125	125-135
Reference	SRT105M1	SRT105M2	SRT105M3	SRT105M4	SRT105M5	SRT105M6

The circumference of the torso is measured

Indications

It is used post surgery or for injuries of the thorax, rib or stern fractures and for intercostal neuralgia.

Description

The orthosis is made of elastic fabric lined on the front side with soft fabric, provided with 2 adjustable shoulder straps, with protection sleeves.

Hook and loop closure on the back.

Width: front 20 cm, back 16 cm.

It is a traditional elastic orthosis that, due to the material, allows normal chest breathing, while ensuring uniform compression on the entire covered surface.

Purpose

The orthosis provides uniform compression on the chest, limits thoracic breathing and transfers a greater percentage to abdominal breathing. In this way, painful symptoms are reduced by limiting thoracic expansion.

Composition

Elastic fabric with a structure that allows high air permeability in both directions.

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The orthosis is placed from the front to the back, where it is fastened with hook and loop. The shoulder straps are adjusted as needed.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**

Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.


In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON	
30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015
SR EN ISO 13485:2016

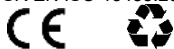

 **Manufacturer SC TRIAMED SRL**

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025