

## OMBIFIX

Abdominal orthosis for umbilical hernia

Ref. SRT 110

### INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

#### Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

#### Available sizes

Sizes	1	2	3	4	5	6
Waist circumference (cm)	65-75	75-85	85-95	95-105	105-115	115-125
Reference	SRT110M1	SRT110M2	SRT110M3	SRT110M4	SRT110M5	SRT110M6

The circumference of the waist is measured

#### Indications

It is used in case of abdominal disorders and umbilical hernia.

#### Description

The orthosis is made of elastic fabric, provided with one semirigid, standard dimension pad (polyurethane foam, Ø 11 cm), positioned on the front, at the level of the navel. Lateral closure with hook and loop straps.

Width: 16 cm.

It is a traditional elastic corset that, due to the material, allows normal thoraco abdominal breathing, while ensuring uniform compression on the entire covered surface. The semirigid pad provides protection and compression to the umbilical region. The pad is removable and can be positioned as needed, with hook and loop. The orthosis can be used in case of umbilical hernia, both before and after surgery.

#### Purpose

The umbilical hernia orthosis provides support and compression to the affected area, reducing pain and preventing hernia aggravation or postoperative recurrence.

#### Composition

Elastic fabric, with a structure that allows high air permeability in both directions.

Abdominal pad: Polyurethane foam lined with soft textile.

#### Positioning

The orthosis is placed from the side, so that the pad is positioned on the navel. It is fastened on the side.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**

#### Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

#### Storage

At room temperature, in a dry space with good ventilation.

#### Commercial warranty of the product: 30 days.

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.


In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

**Deadline for communication of the solution: 15 days.**

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

<b>WARRANTY COUPON</b> 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015  
SR EN ISO 13485:2016

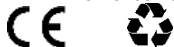

**Manufacturer SC TRIAMED SRL**

**Production** Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

**Registered office** Str. Cosminului nr.12 | Timișoara | Timiș | România

**Phone** 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025