

## ABD 02 TRIA

Post surgery abdominal orthosis

Ref. SRT 109

### INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

#### Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

#### Available sizes

Sizes	1	2	3	4	5	6
Waist circumference (cm)	65-75	75-85	85-95	95-105	105-115	115-125
Reference	SRT109M1	SRT109M2	SRT109M3	SRT109M4	SRT109M5	SRT109M6

The circumference of the waist is measured

#### Indications

It is used in case of abdominal injuries, postoperative or after childbirth.

#### Description

The orthosis is made of elastic fabric, provided with:

- one 6 cm wide elastic strap on the abdomen for a better support
- one abdominal pad inside the orthosis (made of semirigid polyethylene foam, of 18 x 9 cm).

Hook and loop closure on the back.

Width: 27,5 cm.

It is a traditional elastic corset that, due to the material, allows normal thoraco abdominal breathing, while ensuring uniform compression on the entire covered surface.

The semirigid pad, included in the corset, provides protection and compression to the abdominal region. The pad is removable and can be positioned as needed. The orthosis can be used in case of abdominal medical conditions, both before and after surgery.

#### Purpose

The orthosis ensuring support and uniform compression on the entire covered surface.

#### Composition

Elastic fabric, with a structure that allows high air permeability in both directions.

Polyethylene foam lined with soft textile (abdominal pad).

#### Positioning

The orthosis is placed from the front to the back, where it is fastened. The elastic strap of 6 cm goes around the waist and closed on the back, assuring compression on the pad. If lower compression is desired, the pad can be removed.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**

#### Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

#### Storage

At room temperature, in a dry space with good ventilation.

#### Commercial warranty of the product: 30 days.

**Date of production:** see packaging label.

**Validity date:** no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.


In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

#### Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: [suportclienti@triamed.ro](mailto:suportclienti@triamed.ro)

<b>WARRANTY COUPON</b> 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015  
SR EN ISO 13485:2016

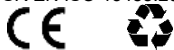

**Manufacturer SC TRIAMED SRL**

**Production** Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

**Registered office** Str. Cosminului nr.12 | Timișoara | Timiș | România

**Phone** 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025