

AKTIV LIGHT 01 TRIA

Mobile knee orthosis with patellar support

Ref. SRT 308

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes

| Sizes | 1 | 2 | 3 | 4 | 5 | 6 | 7 | 8 |
|-------------------------|----------|----------|----------|----------|----------|----------|----------|----------|
| Knee circumference (cm) | 33-35 | 35-37 | 37-39 | 39-41 | 41-43 | 43-45 | 45-47 | 47-49 |
| Reference | SRT308M1 | SRT308M2 | SRT308M3 | SRT308M4 | SRT308M5 | SRT308M6 | SRT308M7 | SRT308M8 |

The circumference of the knee is measured at the middle of the knee cap

The orthosis can be used on both the left and right knee.

Indications

It is used in cricks, luxations, post consolidated fractures, inflammatory processes, lateral knee joint instability. It is recommended both as orthopedic treatment and postoperative.

Description

Tubular, closed orthosis, made of textile, provided with:

- a non-slip silicone elastic band (10mm) inside the upper part
- 2 straps for fixation provided with elastic hook and loop fastener, that mold to the conical anatomy of the thigh, preventing the orthosis from slipping
- 4 flexible metal splints, 2 on each side
- patellar support (polyethylene foam).

Length: 29 cm.

Due to its tubular, sleeve shape, it is not recommended to be used in case of oedema or significant swelling.

Purpose

The orthosis is a support for the knee joint, the 2 flexible metal splints for each side providing stability and safety in use.

Composition


Neoprene coated on both sides with fabric.

Positioning

The orthosis is placed on the knee so that the patella support surrounds the knee cap and the lateral splints are parallel with the median axis of the leg. The 2 straps are passed through buckles and are fixed to the optimal compression.

If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**

Cleaning

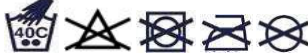

To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.

Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

| | |
|---|-------------------------------|
| WARRANTY COUPON 30 days from the date of purchase | Batch and date of manufacture |
| Product reference | Date of purchase |
| The reason of complaint | Seller name and signature |

SR EN ISO 9001:2015

SR EN ISO 13485:2016



Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului.nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/31.07.2025