

TRIAFIX ELBOW

Fixed elbow wrist hand orthosis

Ref. SRT 214

INSTRUCTIONS FOR USE

Dear Customer!

Thank you for the confidence you have shown in purchasing Triamed products! We would like to inform you that Triamed products are made of the best performing materials, all having quality certificates and biocompatibility tests. These orthoses comply with the Technical Specification and the requirements of Regulation (EU) MDR 2017/745 of the European Parliament on medical devices and comply with the requirements set out in Annex I - General Requirements for safety and performance- and Annex VIII - Classification rules. Conformity is declared on the basis of Article 52 (7), Annex II and Annex III of MDR 2017/745.

Important information

The orthosis is used according to the recommendations of the specialized physician.

Choosing the size - For optimal results, the appropriate size will be selected based on the measurements table of the product. A size too small may lead to intolerance to wearing or the damage of the product! A size too large significantly diminishes the efficacy of the orthosis.

This orthosis should only be applied in accordance with the instructions mentioned here. The product is intended for single patient use.

No liability is accepted for incorrect product use. Guarantee is bound to statutory regulations.

Available sizes, left/right versions

Sizes	1	2	3	4	5	6
Elbow circumference (cm)	22-24	24-26	26-28	28-30	30-32	32-34
Forearm length (cm)	22-25	22-25	25-28	25-28	28-31	28-31
Reference	SRT214M1	SRT214M2	SRT214M3	SRT214M4	SRT214M5	SRT214M6

After the code must be added **L** for Left, **R** for Right

The circumference of the elbow and the length of the forearm are measured

Indications

It is used in tendinopathy, carpal tunnel syndrome, radial nerve paralysis, cubital tunnel syndrome, fracture at the level of the wrist or elbow, both as orthopedic treatment and postoperative.

Description

Orthosis made of textile, provided with:

- one mouldable metal palm splint
- 2 fixed metal elbow splints set at 90°, mouldable, of the same length as the orthosis
- 3 straps with hook and loop (2 on the forearm and one on the arm)
- adjustable support strap (with "crocodile" type fastener and protection sleeve), optional. The positioning is done on the first fixing strap on the forearm and over the opposite shoulder.

Hook and loop fastener on the dorsal side of the hand.

Length: 51 - 57 cm.

Purpose

The orthosis stabilizes and immobilizes the elbow and the radiocarpal joint.

Composition

Laminated textile. The laminated textile ensemble behaves like a moisture regulator, ensuring a quick transfer of the moisture to outward.

Positioning

The orthosis is positioned so that one metal splint supports the palm and the other 2 frame the elbow. The orthosis is fastened with the 3 straps to the optimal compression. If it is necessary to adjust the position of the orthosis, the adjustment is made by fully opening the hook and loop fastener and repositioning the orthosis higher or lower on the body segment, depending on the need. **To avoid the damage of the material, do not pull the orthosis while is fixed.**

Cleaning



To be washed by hand with warm water, no bleaching, no spinning, no ironing, no chemical cleaning!

Storage

At room temperature, in a dry space with good ventilation.

Commercial warranty of the product: 30 days.

Date of production: see packaging label.

Validity date: no expiry date.

To validate the warranty, the seller must indicate the date of purchase. The buyer is obliged to keep and present the warranty coupon.

In order to solve the problems that appeared in the warranty, the following data must be notified: Article reference, batch and date of manufacture, indicated on the packaging box of the product.


Deadline for communication of the solution: 15 days.

Please send notifications and complaints to the following email address: suportclienti@triamed.ro

WARRANTY COUPON 30 days from the date of purchase	Batch and date of manufacture
Product reference	Date of purchase
The reason of complaint	Seller name and signature

SR EN ISO 9001:2015

SR EN ISO 13485:2016

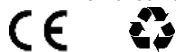

Manufacturer SC TRIAMED SRL

Production Calea Buziaşului nr. 77, Hall 1-C1/A | Timișoara | Timiș | România

Registered office Str. Cosminului.nr.12 | Timișoara | Timiș | România

Phone 00 40 256 487388

office@triamed.ro | triamed@gmail.com | www.triamed.ro

Edition 6, Revision 0/28.07.2025